

TEST CHANGE

Secretin

0099772, SECRETIN

Specimen Requirements:

Patient Preparation: Patient should be fasting for 10-12 hours prior to collection of specimen. Medications affecting intestinal motility or insulin levels should be discontinued, if possible 48 hours prior to collection.

Collect: GI preservative tube (ARUP supply #47531). Available online through eSupply using ARUP Connect (TM) or contact ARUP Client Services at (800-)522-2787.

Specimen Preparation: Separate from cells within 10 minutes. Transfer 4 mL plasma to an ARUP standard transport tube and freeze immediately. (Min: 1 mL) Test is not performed at ARUP; separate specimens must be submitted when multiple tests are ordered.

Transport Temperature: CRITICAL FROZEN.

Unacceptable Conditions:

Remarks:

Stability: Ambient: Unacceptable; Refrigerated: Unacceptable; Frozen: 6 months

Methodology: Quantitative Radioimmunoassay (RIA)

Performed: Varies

Reported: ~~15-20~~10-13 days

Note:

CPT Codes: 83519

New York DOH Approval Status: Specimens from New York clients will be sent out to a New York DOH approved laboratory, if possible.

Interpretive Data:

Reference Interval:

By report

