

TEST CHANGE

Allergen, Food, Gelatin Porcine IgE

2001913, GEL PORC

Specimen Requirements:

Patient Preparation:

Collect:

Plain red or serum separator tube- ~~(SST)~~.

Specimen Preparation:

Separate serum from cells ASAP or within 2 hours of collection. Transfer 0.5 mL serum plus 0.1 mL for each additional allergen ordered to an ARUP ~~Standard Transport Tube-standard transport tube~~. (Min: 0.34 mL plus 0.04 mL for each allergen ordered)

Test is not performed at ARUP; separate specimens must be submitted when multiple tests are ordered.

Transport Temperature:

~~Room temperature-Frozen~~. Also acceptable: ~~RefrigeratedRoom temperature~~ or ~~frozenrefrigerated~~.

Unacceptable Conditions:

~~Hemolyzed, icteric, or lipemic specimens-~~

Remarks:

Stability:

Ambient: 1 month; Refrigerated: 1 month; Frozen: 1 year

Methodology:

Quantitative Enzyme Immunoassay (EIA)

Note:

CPT Codes:

86003

New York DOH Approval Status:

This test is New York DOH approved.

Interpretive Data:

Reference Interval:

Refer to report