

2002357

JAK2 Exon 12 Mutation Analysis by PCR

JAK2 EX12

**Specimen Required:** Collect: **Whole blood** or bone marrow (EDTA). Also acceptable: DNA extracted by CLIA-certified lab.

Specimen Preparation: **Whole Blood:** Do not freeze. Transport 5 mL whole blood. (Min: 1 mL)

**Bone Marrow:** Do not freeze. Transport 3 mL bone marrow. (Min: 1 mL)

**Extracted DNA:** Transport 40 uL DNA with at least 50 ng/uL concentration. (Min: 40 uL) Transport DNA in a tissue transport kit (ARUP Supply #47808) available online through eSupply using ARUP Connect™ or contact ARUP Client Services at (800) 522-2787.

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Plasma, serum, FFPE tissue blocks/slides, or frozen **tissue. Specimens** collected in anticoagulants other than **EDTA. Clotted** or grossly hemolyzed specimens.

Stability (collection to initiation of testing): Ambient: 24 hours; Refrigerated: 5 days; Frozen: Unacceptable

**Extracted DNA:** Ambient: 1 month; Refrigerate: Indefinitely; Frozen: Indefinitely