

HOTLINE: Effective May 17, 2021

2003020

Human Epididymis Protein 4 (HE4)

HE4

Specimen Required: Collect: Serum Separator Tube (SST). Also acceptable: Green (Lithium Heparin), Lavender (K₂ EDTA), or Lavender (K₃ EDTA).
Specimen Preparation: Allow specimen to clot completely at room temperature. Transfer 1 mL serum or plasma to an ARUP Standard Transport Tube. (Min: 0.5 mL)
Storage/Transport Temperature: Frozen.
Unacceptable Conditions: Grossly hemolyzed specimens.
Stability (collection to initiation of testing): Ambient: 5 hours; Refrigerated: 48 hours; Frozen: 3 months