

## HOTLINE: Effective November 14, 2022

## 2003347 Rheumatoid Factor, Body Fluid

**RA-FL** 

Specimen Required: Collect: CSF, Pericardial, Pleural, or Synovial fluid.

New York State Clients: Pericardial fluid requires NY clients to submit a Non-Permitted Laboratory Request (NPL)

form to NYSDOH.

Specimen Preparation: Centrifuge to remove cellular material. Transfer 1 mL body fluid to an ARUP Standard Transport Tube. (Min:

0.5 mL)

<u>Storage/Transport Temperature:</u> Refrigerated. <u>Remarks:</u> **Specimen source must be provided.** 

<u>Unacceptable Conditions:</u> Specimen types other than those listed. Specimens too viscous to be aspirated by instrument.

Stability (collection to initiation of testing): Ambient: 24 hours; Refrigerated: 1 week; Frozen: 1 month (should not be thawed more

than once)

Private Information Page 1