
2005103 Oxycodone/Oxymorphone Screen Only, Urine

OXY SCR UR

Specimen Required: Collect: Random urine.

Specimen Preparation: Transfer 4 mL urine with no additives or preservatives to an ARUP Standard Transport Tube. (Min: 2 mL)

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Specimens exposed to repeated freeze/thaw cycles.

Stability (collection to initiation of testing): Ambient: 1 week; Refrigerated: 1 month; Frozen: **1 month**