

## TEST CHANGE

### Baclofen Quantitative, Serum or Plasma

2005273, BACLO SP

#### Specimen Requirements:

##### Patient Preparation:

Collect: Plain red, lavender (K2EDTA), or pink (K2EDTA).

Specimen Preparation: Separate from cells ASAP or within 2 hours of collection. Transfer 1 mL serum or plasma to an ARUP standard transport tube. (Min: 0.4 mL)  
Test is not performed at ARUP; separate specimens must be submitted when multiple tests are ordered.

Transport Temperature: ~~Frozen~~Refrigerated. Also acceptable: Room temperature or refrigeratedfrozen.

Unacceptable Conditions: ~~Polymer gel separation tube (SST or PST). Separator tubes.~~

##### Remarks:

Stability: Ambient: 2 weeks; Refrigerated: 2 weeks; Frozen: 4 months

Methodology: Quantitative High Performance Liquid Chromatography-Tandem Mass Spectrometry

##### Note:

CPT Codes: 80369 ~~(Alt code: G0480)~~

New York DOH Approval Status: This test is New York DOH approved.

#### Interpretive Data:

#### Reference Interval:

Refer to report