

---

**2006267****Cytogenomic SNP Microarray Buccal Swab****CMA BUCCAL**

**Specimen Required:** Collect: One buccal swab using the OraCollect collection kit ensuring the sponge tip does not come into contact with any surface prior to collection. Donor should not eat, drink, smoke or chew gum for 30 minutes before collecting oral sample.

Specimen Preparation: Transport Buccal swab in ORACollect Collection kit (ARUP supply #49295). Available online through eSupply using ARUP Connect™ or contact ARUP Client Services at (800) 522-2787.

Storage/Transport Temperature: Room temperature.

Unacceptable Conditions: Specimen exposed to extreme temperatures. Specimens collected in or by any specimen device other than indicated.

Stability (collection to initiation of testing): Ambient: **7 days**; Refrigerated: **Unacceptable**; Frozen: **Unacceptable**