

---

2006685

**Thyroglobulin, Serum or Plasma with Reflex to LC-MS/MS or CIA**

**THYROGRFX**

**Specimen Required:** Collect: Serum Separator Tube (SST). Also acceptable: Green (Sodium or Lithium Heparin).

Specimen Preparation: Transfer 3 mL serum or plasma to an ARUP Standard Transport Tube. (Min: 1.5 mL)

Storage/Transport Temperature: Frozen.

Unacceptable Conditions: EDTA or K2EDTA plasma. Grossly lipemic samples. Any fluid other than noted above. (See Thyroglobulin, Fine Needle Aspiration (FNA) (ARUP test code 0020753) for ordering alternate fluids).

Stability (collection to initiation of testing): After separation from cells: Ambient: 8 hours; Refrigerated: 1 week; Frozen: 6 months