

Quarterly HOTLINE: Effective February 19, 2019

2007945 Aripiprazole and Metabolite, Serum or Plasma

ARIPIIPRAZO

Performed: Wed, Sat
Reported: 1-5 days

Specimen Required: Patient Prep: Pre-dose (trough) draw - At steady state concentration.

Collect: Plain Red. Also acceptable: Lavender (EDTA) or Pink (K₂EDTA).

Specimen Preparation: **Separate from** cells ASAP or within 2 hours of collection. Transfer 1 mL serum or plasma to an ARUP Standard Transport Tube. (Min: 0.5 mL)

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Gel separator tubes, light blue (citrate), or yellow (SPS or ACD solution).

Stability (collection to initiation of testing): Ambient: 2 weeks; Refrigerated: 2 weeks; Frozen: 2 weeks

HOTLINE NOTE: Remove information found in the Specimen Required Remarks field. There is also a component change associated with this test.

Remove component 2011432, Aripiprazole Dose

Remove component 2011433, Aripiprazole Dose Frequency

Remove component 2011434, Aripiprazole Route

Remove component 2011435, Aripiprazole Type of Draw