

HOTLINE: Effective June 7, 2021

2007945

Aripiprazole and Metabolite, Serum or Plasma

ARIPIIPRAZO

**Specimen Required:** Patient Prep: Pre-dose (trough) draw - At steady state concentration.

Collect: Plain Red. Also acceptable: Lavender (EDTA) or Pink (K<sub>2</sub>EDTA).

Specimen Preparation: Separate from cells ASAP or within 2 hours of collection. Transfer 1 mL serum or plasma to an ARUP Standard Transport Tube. (Min: 0.5 mL)

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: **Whole blood.** Gel separator tubes, light blue (citrate), or yellow (SPS or ACD solution).

Stability (collection to initiation of testing): Ambient: 2 weeks; Refrigerated: 2 weeks; Frozen: 2 weeks

**Reference Interval:**

Effective June 7, 2021

Therapeutic Range (Aripiprazole and Dehydroaripiprazole)	150-500 ng/mL
Toxic range (Aripiprazole and Dehydroaripiprazole)	Greater than or equal to 1000 ng/mL