

---

**2008320**

**Infliximab or Biosimilar Activity and Neutralizing Antibody**

**IFX NAB**

**Specimen Required:** Patient Prep: Collect specimens before infliximab or biosimilar treatment.

Collect: Serum separator tube.

Specimen Preparation: Separate serum from cells ASAP or within 2 hours of collection. Transfer 1 mL serum to an ARUP Standard Transport Tube. (Min: 0.5 mL)

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Contaminated, hemolyzed, icteric, or lipemic specimens.

Stability (collection to initiation of testing): After separation from cells: Ambient: 48 hours; Refrigerated: 4 weeks; Frozen: 1 year (avoid repeated freeze/thaw cycles)