

TEST CHANGE

Hepatitis E Virus (HEV) Antibody, IgM

2010156, HEV IGM

Specimen Requirements:

Patient Preparation:

Collect: Serum separator tube. Also acceptable: Red (clot activator), lavender (EDTA), pink (K2EDTA).

Specimen Preparation: Separate from cells ASAP or within 2 hours of collection. Transfer 0.5 mL serum to an ARUP [standard transport tube](#). [Standard Transport Tube](#). (Min: 0.3 mL)

Transport Temperature: Refrigerated.

Unacceptable Conditions: Specimens containing particulate material.

Remarks:

Stability: After separation from cells: Ambient: [48 hours](#)~~Unacceptable~~; Refrigerated: [14 days](#)~~2 weeks~~; Frozen: [30 days](#)~~Indefinitely~~ (avoid repeated freeze/thaw cycles)

Methodology: Qualitative Enzyme-Linked Immunosorbent Assay (ELISA)

Note:

CPT Codes: 86790

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

~~This test was developed and its performance characteristics determined by ARUP Laboratories. It has not been cleared or approved by the US Food and Drug Administration. This test was performed in a CLIA-certified laboratory and is intended for clinical purposes.~~

Reference Interval:

Test Number	Components	Reference Interval
	Hepatitis E Virus Ab, IgM	Negative

Deleted Cells

Deleted Cells