

HOTLINE: Effective August 19, 2019

2011075	<i>Coccidioides</i> Antigen Quantitative by EIA	COCCI AG
<b>Performed:</b>	Varies	
<b>Reported:</b>	3-4 days	
<b>Specimen Required:</b>		
Collect: Plain Red, Serum Separator Tube (SST), Lavender (K <sub>2</sub> or K <sub>3</sub> EDTA), Pink (K <sub>2</sub> EDTA), Green (Sodium or Lithium Heparin), or Light Blue (CTAD). Also acceptable: Urine, CSF, or BAL.		
Specimen Preparation: Transfer 2 mL serum or plasma to an ARUP Standard Transport Tube. (Min: 1.2 mL)		
Transfer 1 mL urine or BAL to an ARUP Standard Transport Tube. (Min: 0.5 mL)		
Transfer 1 mL CSF to an ARUP Standard Transport Tube. (Min: 0.8 mL)		
Storage/Transport Temperature: Refrigerated. Also acceptable: Room temperature or frozen.		
Remarks: Specimen source required.		
Stability (collection to initiation of testing): Ambient: 28 days; Refrigerated: 28 days; Frozen: Indefinitely		