
2011075

***Coccidioides* Antigen Quantitative by EIA**

COCCI AG

Specimen Required: Collect: Plain Red, Serum Separator Tube (SST), Lavender (K₂ or K₃EDTA), Pink (K₂EDTA), Green (Sodium or Lithium Heparin), or Light Blue (CTAD). Also acceptable: Urine, CSF, or BAL.

Specimen Preparation: Transfer 2 mL serum or plasma to an ARUP Standard Transport Tube. (Min: 1.2 mL)

Transfer 1 mL urine or BAL to an ARUP Standard Transport Tube. (Min: 0.5 mL)

Transfer 1 mL CSF to an ARUP Standard Transport Tube. (Min: 0.8 mL)

Storage/Transport Temperature: Refrigerated. Also acceptable: Room temperature or frozen.

Remarks: Specimen source required.

Stability (collection to initiation of testing): Ambient: **2 weeks**; Refrigerated: **2 weeks**; Frozen: Indefinitely