

---

2011117

**Myeloid Malignancies Mutation Panel by Next Generation Sequencing**

**MYE NGS**

**Specimen Required:** Collect: Lavender (EDTA) **OR** Bone Marrow (EDTA).

Specimen Preparation: Do not freeze. Transport 5 mL whole blood. (Min: 1 mL) **OR** Transport 3 mL bone marrow. (Min: 1 mL).

**Separate specimens must be submitted when multiple tests are ordered.**

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Serum, plasma or tissue. Specimens collected in anticoagulants other than EDTA. Clotted or grossly hemolyzed specimens.

Stability (collection to initiation of testing): Ambient: 24 hours; Refrigerated: 5 days; Frozen: Unacceptable