

Quarterly HOTLINE: Effective February 19, 2019

2011609	Pregabalin, Serum or Plasma	PREGABALIN
---------	-----------------------------	------------

Performed: Wed, Sat
Reported: 1-6 days

Specimen Required: Collect: Serum Pre-dose (Trough) Draw - At a Steady State Concentration or Plasma Pre-dose (Trough) Draw - At a Steady State Concentration in Plain Red, Lavender (K₂EDTA), Lavender (K₃EDTA), or Pink (K₂EDTA).

Specimen Preparation: Separate from cells ASAP or within 2 hours of collection. Transfer 1 mL serum or plasma to an ARUP Standard Transport Tube. (Min: 0.2 mL)

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Citrated Plasma.

Stability (collection to initiation of testing): Ambient: 1 month; Refrigerated: 1 month; Frozen: 2 months

HOTLINE NOTE: Remove information found in the Specimen Required Remarks field. There is also a component change associated with this test.

Remove component 2011894, Pregabalin Dose

Remove component 2011895, Pregabalin Route

Remove component 2011896, Pregabalin Dose Frequency

Remove component 2011897, Pregabalin Type of Draw