

HOTLINE: Effective May 16, 2022

---

2011808

Chikungunya Antibody, IgG

CHIKG

**Specimen Required:** Collect: Serum or plasma (heparin, citrate, or EDTA)

Specimen Preparation: Separate serum from cells ASAP or within 2 hours of collection. Transfer 1 mL serum to an ARUP Standard Transport Tube. (Min: 0.15 mL) Parallel testing is preferred and convalescent specimens **must** be received within 30 days from receipt of the acute specimens. **Mark specimens plainly as "acute or convalescent."**

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Contaminated, heat-inactivated, hemolyzed, or severely lipemic specimens.

Stability (collection to initiation of testing): After separation from cells: Ambient: 48 hours; Refrigerated: 2 weeks; Frozen: 1 year (avoid repeated freeze/thaw cycles)