

Effective Date: July 21, 2025

TEST CHANGE

Chikungunya Antibodies, IgG and IgM

2011812, CHIKPAN

Reference Interval:

ZUTTOTZ, CHIIKFAIN	
Specimen Requirements:	
Patient Preparation:	
Collect:	<u>Serumseparator tube (SST). Also acceptable: Plain redSerum</u> or plasma (heparin, citrate, or EDTDA).
Specimen Preparation:	Separate serum from cells ASAP or within 2 hours of collection. Transfer 1 mL serum to an ARUP standard transport tubeStandard Transport Tube. (Min: 0.15 mL) Parallel testing is preferred and convalescent specimens must be received within 30 days from receipt of the acute specimens. Mark specimens plainly as "acute or convalescent."
Transport Temperature:	Refrigerated
Unacceptable Conditions:	Contaminated, heat-inactivated, hemolyzed, or severely lipemic specimens.
Remarks:	
Stability:	After separation from cells: Ambient: 48 hours; Refrigerated: 2 weeks; Frozen: 1 year (avoid repeated freeze/thaw cycles)
Methodology:	Semi-Quantitative Enzyme-Linked Immunosorbent Assay
Performed:	Wed
Reported:	1-8 days
Note:	
CPT Codes:	86790 x2
New York DOH Approval Status:	This test is New York DOH approved.
Interpretive Data:	

This test was developed and its performance characteristics determined by ARUP Laboratories. It has not been cleared or approved by the US Food and Drug Administration. This test was

Deleted Cells

performed in a CLIA certified laboratory and is intended for clinical purposes.

Chikungunya Antibody, IgM

2011810

Test Code	Component	Reference Interval	
2011808	Chikungunya Antibody, IgG	0.79 Index or less	Negative: No significant level of Chikungunya IgG antibody detected.
		0.80-1.09 Index	Equivocal: Questionable presence of Chikungunya IgG antibody detected. Repeat testing in 10-14 days may be helpful.

1.10 Index or

0.80-1.09 Index

1.10 Index or greater

greater

Positive:

0.79 Index or less significant level of Chikungunya IgM antibody detected.

Chikungunya IgG antibody detected; suggests current or past infection.

etected.
Equivocal:
Questionable
presence of
Chikungunya IgM
antibody
detected. Repeat
testing in 10-14
days may be
helpful.

Positive: Chikungunya IgM

antibody detected.

Effective Date: July 21, 2025