
2012225

Benzodiazepines, Urine Screen with Reflex to Quantitation

BENZ RFX U

Specimen Required: Collect: Random urine.

Specimen Preparation: Transfer 4 mL urine with no additives or preservatives to an ARUP Standard Transport Tube. (Min: 2 mL)

Storage/Transport Temperature: Room Temperature

Unacceptable Conditions: Breast Milk. Pharmaceutical preparation. Specimens exposed to repeated freeze/thaw cycles.

Stability (collection to initiation of testing): Ambient: 1 week; Refrigerated: 1 month; Frozen: **1 month**