
2013008

Periprosthetic Joint Infection (PJI) Detection (Synovasure)

SYNOVA PJI

Specimen Required: Collect: Synovial fluid in Plain Red. **Due to limited stability, only collect Monday-Wednesday and not the day before a holiday.**
Specimen Preparation: Transport 1 mL synovial fluid in the original collection tube. (Min: 0.5 mL) **Send the same day as collection.**
Due to limited stability, only send Monday-Wednesday and not the day before a holiday.
Test is not performed at ARUP; separate specimens must be submitted when multiple tests are ordered.
Storage/Transport Temperature: Refrigerated.
Remarks: Anatomical source is required (eg, left knee, right shoulder, etc.).
Stability (collection to initiation of testing): Ambient: 72 hours; Refrigerated: 4 days; Frozen: Unacceptable