

HOTLINE: Effective September 7, 2021

---

2013560

C-Peptide, 60 Minutes

C PEP 60

**Specimen Required:** Patient Prep: Fasting specimen preferred.

Collect: Serum Separator Tube (SST)

Specimen Preparation: Allow specimen to clot completely at room temperature. Separate from cells ASAP or within 2 hours of collection. Transport 1 mL serum to an ARUP Standard Transport Tube. (Min: 0.5 mL)

Storage/Transport Temperature: Frozen.

Unacceptable Conditions: Grossly hemolyzed specimens.

Stability (collection to initiation of testing): After separation from cells: Ambient: 8 hours; Refrigerated: 7 days; Frozen: 1 month