
2013990

Polymyositis Panel

POLY MYO

Specimen Required: Collect: Serum Separator Tube (SST).

Specimen Preparation: Separate from cells ASAP or within 2 hours of collection. Transfer two 1 mL serum aliquots to ARUP

Standard Transport Tubes. (Min: 0.5 mL/aliquot)

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Hemolyzed, hyperlipemic, icteric, heat-treated or contaminated specimens.

Stability (collection to initiation of testing): Ambient: 48 hours; Refrigerated: 2 weeks; Frozen: **1 month**