

**TEST CHANGE**

Allergen, Fungi and Molds, *Aspergillus flavus* IgE

2014003, ASPER FLA

Specimen Requirements:

Patient Preparation:

Collect: Plain ~~r~~Red or serum separator tube~~Serum Separator Tube~~ (SST).

Specimen Preparation: Separate from cells ASAP or within 2 hours of collection. Transfer 0.5 mL serum plus 0.1 mL for each additional allergen ordered to an ARUP standard transport tube~~Standard Transport Tube~~. (Min: 0.34 mL plus 0.04 mL for each allergen ordered)  
Test is not performed at ARUP; separate specimens must be submitted when multiple tests are ordered.

Transport Temperature: ~~Frozen. Room temperature.~~ Also acceptable: Room temperature~~Refrigerated~~ or refrigerated~~frozen~~.

Unacceptable Conditions: Lipemic specimens.

Remarks:

Stability: Ambient: 1 month; Refrigerated: 1 month; Frozen: 1 year

Methodology: Quantitative Enzyme Immunoassay (EIA)

Note:

CPT Codes: 86003

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Reference Interval:

Refer toBy report