

TEST CHANGE

Allergen, Fungi and Molds, *Fusarium solani* IgE

2014005, FUSARIUM

Specimen Requirements:

Patient Preparation:

Collect: Plain Red or ~~serum separator tube~~ ~~Serum Separator Tube~~ (SST).

Specimen Preparation: Separate from cells ASAP or within 2 hours of collection. Transfer 0.5 mL serum plus 0.1 mL for each additional allergen ordered to an ARUP ~~standard transport tube~~ ~~Standard Transport Tube~~. (Min: 0.34 mL plus 0.04 mL for each allergen ordered)
Test is not performed at ARUP; separate specimens must be submitted when multiple tests are ordered.

Transport Temperature: ~~Frozen~~ ~~Room temperature~~. Also acceptable: ~~Room temperature~~ ~~Refrigerated~~ or ~~refrigerated~~ ~~frozen~~.

Unacceptable Conditions: ~~Lipemic specimens~~

Remarks:

Stability: Ambient: 1 month; Refrigerated: 1 month; Frozen: 1 year

Methodology: Quantitative Enzyme Immunoassay ~~(EIA)~~

Note:

CPT Codes: 86003

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Reference Interval:

~~Refer to~~ ~~By~~ report