

TEST CHANGE

Prostate-Specific Kallikrein, 4Kscore

2014059, 4KSCORE

Specimen Requirements:

Patient Preparation:

Collect: Serum separator tube (SST).

Specimen Preparation: Transfer 4 mL serum to an ARUP standard transport tube. (Min: 3 mL) Test is not performed at ARUP; separate specimens must be submitted when multiple tests are ordered.

Transport Temperature: Frozen.

Unacceptable Conditions: Frozen serum separator tubes (SST).

Remarks: Biopsy history must be provided at time of order.

Stability: Ambient: Unacceptable; Refrigerated: 72 hours; Frozen: 1 month

Methodology: Electrochemiluminescent Immunoassay (ECLIA)

Performed: Varies

Reported: ~~5-10~~3-6 days

Note: 4 Kallikrein Biomarkers: Total PSA, free PSA, percent free PSA, intact PSA and hK2. A digital rectal exam (DRE) is required and submissions should indicate "nodule" or "no nodule." Test should not be ordered if DRE has been performed within the last 4 days or if biopsy history is positive. DREs performed after collection of specimen are acceptable.

CPT Codes: 81539

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Reference Interval:

By report



*A nonprofit enterprise of the University of Utah
and its Department of Pathology*

Effective Date: **August 5, 2024**