

Quarterly HOTLINE: Effective June 26, 2018

---

<b>New Test</b> Available Now	<b>3000503</b>	<b>Cortisol, Right Adrenal Vein</b>	<b>CORT RAV</b>
----------------------------------	----------------	-------------------------------------	-----------------

---

**Methodology:** Quantitative Chemiluminescent Immunoassay  
**Performed:** Sun-Sat  
**Reported:** Within 24 hours

**Specimen Required:** Patient Prep: Adrenal venous sampling procedure is required.

Collect: Serum Separator Tube (SST).

Specimen Preparation: Allow specimen to clot completely at room temperature. Separate from cells ASAP or within 2 hours of collection. Transfer 1 mL serum to an ARUP Standard Transport Tube. (Min: 0.4 mL)

Storage/Transport Temperature: Refrigerated.

Stability (collection to initiation of testing): After separation from cells: Ambient: 1 week; Refrigerated: 1 week; Frozen: 3 months

**Note:** To convert to nmol/L, multiply  $\mu\text{g/dL}$  by 27.6.

**CPT Code(s):** 82533

New York DOH Approved.

**HOTLINE NOTE:** Refer to the Test Mix Addendum for interface build information.