

TEST CHANGE

Mumps Virus by PCR

3000523, MPSPCR

Specimen Requirements:

| | |
|--------------------------|---|
| Patient Preparation: | Patient should not eat, drink, smoke or chew gum for 30 minutes before collecting oral sample. |
| Collect: | Buccal swab. |
| Specimen Preparation: | Transfer buccal swab to viral transport media (ARUP supply #12884) available online through eSupply using ARUP Connect(TM) or contact ARUP Client Services at (800) 522-2787. (Min: 0.5 mL) |
| Transport Temperature: | Frozen. |
| Unacceptable Conditions: | Urine. Nasopharyngeal swab. |
| Remarks: | Specimen source required. |
| Stability: | Ambient: 48 hours; Refrigerated: 1 week; Frozen: 1 week |
| Methodology: | Qualitative Polymerase Chain Reaction |

Note:

CPT Codes: 87798

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

This test was developed and its performance characteristics determined by ARUP Laboratories. It has not been cleared or approved by the US Food and Drug Administration. This test was performed in a CLIA certified laboratory and is intended for clinical purposes.

Reference Interval:

HOTLINE NOTE: There is a price change associated with this test. Please contact ARUP Client Services at (800) 522-2787 for additional information.