

Quarterly HOTLINE: Effective February 19, 2019

---

<b>New Test</b>	<b>3001311</b>	<b><i>BCL6 (3q27) Gene Rearrangement by FISH</i></b>	<b>BCL6_FISH</b>
-----------------	----------------	--	------------------

---



Additional Technical Information

**Methodology:** Fluorescence in situ Hybridization  
**Performed:** Monday-Friday  
**Reported:** 3-7 days

**Specimen Required:** Collect: Tumor tissue.

Specimen Preparation: Formalin fix (10 percent neutral buffered formalin) and paraffin embed specimen. Protect paraffin block and/or slides from excessive heat. Transport tissue block or 5 unstained (3-micron thick sections), positively charged slides in a tissue transport kit (ARUP supply #47808 recommended but not required), available online through eSupply using ARUP Connect™ or contact ARUP Client Services at (800) 522-2787. (Min: 2 slides) If sending precut slides, do not oven bake.

Storage/Transport Temperature: Room temperature or refrigerated. Ship in cooled container during summer months.

Remarks: Include surgical pathology report.

Unacceptable Conditions: Specimens fixed or processed in alternative fixatives (alcohol, Prefer) or heavy metal fixatives (B-4 or B-5). No tumor in tissue. Decalcified specimens.

Stability (collection to initiation of testing): Ambient: Indefinitely; Refrigerated: Indefinitely; Frozen: Unacceptable

**Interpretive Data:**

Refer to report.

See Compliance Statement A: [www.aruplab.com/CS](http://www.aruplab.com/CS)

**CPT Code(s):** 88366

New York DOH Approved.

**HOTLINE NOTE:** Refer to the Test Mix Addendum for interface build information.