

New Test
Available Now

3001410

Basement Membrane Zone Antibody Panel

BMZ AB PAN

Methodology: Indirect Fluorescent Antibody/Enzyme-Linked Immunosorbent Assay

Performed: Varies

Reported: 4-9 days

Specimen Required: Collect: Plain Red or Serum Separator Tube (SST).

Specimen Preparation: Transfer 2 mL serum to an ARUP Standard Transport Tube. (Min: 0.5 mL)

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Plasma. Hemolyzed or lipemic specimens.

Stability (collection to initiation of testing): Ambient: 1 week; Refrigerated: 2 weeks; Frozen: Indefinitely

Reference Interval: By report

Interpretive Data: Refer to report.

Note: For specimens less than 0.5 mL, call the Immunodermatology Laboratory at (866) 266-5699.

CPT Code(s): 88346, 88350 x3, 83516 x3

New York DOH approval pending. Call for status update.

HOTLINE NOTE: Refer to the Test Mix Addendum for interface build information.