

TEST CHANGE

CYP3A4 and CYP3A5

3001518, 3A4/3A5

Specimen Requirements:

Patient Preparation:

Collect: Lavender (EDTA), pink (K2EDTA), or yellow (ACD solution A or B).

Specimen Preparation: Transport 3 mL whole blood. (Min: 1 mL)

Transport Temperature: Refrigerated.

Unacceptable Conditions: Plasma or serum. Specimens collected in sodium heparin or lithium heparin. Frozen specimens in glass collection tubes.

Remarks:

Stability: Ambient: 72 hours; Refrigerated: 1 week; Frozen: 1 month

Methodology: Polymerase Chain Reaction (PCR)/Fluorescence Monitoring

Performed: Varies

Reported: 5-10 days

Note:

CPT Codes: 81230; 81231

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Refer to report

Counseling and informed consent are recommended for genetic testing. Consent forms are available online.

Reference Interval:

By report

HOTLINE NOTE: There is a component change associated with this test. One or more components have been added or removed. Refer to the Hotline Test Mix for interface build information.