
3001541

Warfarin Sensitivity (*CYP2C8, CYP2C9, CYP4F2, VKORC1*) Genotyping

WARF PAN

Specimen Required: Collect: Lavender (EDTA), Pink (K₂EDTA), or Yellow (ACD Solution A or B).

Specimen Preparation: Transport 3 mL whole blood. (Min: 1 mL) OR Transport the Saliva Collection Device.

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Plasma or serum. Specimens collected in sodium heparin or lithium heparin. Frozen specimens in glass collection tubes.

Stability (collection to initiation of testing): Ambient: 72 hours; Refrigerated: 1 week; Frozen: 1 month