
3001798

Blastomyces Antigen Quantitative by EIA, Urine

BLASTOAG U

Specimen Required: Collect: Urine.

Specimen Preparation: Transfer 1 mL urine to an ARUP Standard Transport Tube. (Min: 0.5 mL)

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Specimens preserved in boric acid.

Stability (collection to initiation of testing): Ambient: **Unacceptable**; Refrigerated: 2 weeks; Frozen: Indefinitely