

HOTLINE: Effective May 18, 2020

3002008	Human Papillomavirus (HPV) High Risk by in situ Hybridization, Paraffin	HPVHR ISH
----------------	--	------------------

Specimen Required: Collect: Tissue.

Specimen Preparation: Formalin fix (10 percent neutral buffered formalin) and paraffin-embed tissue. Transport tissue block or 5 unstained 5-micron slides in a tissue transport kit (recommended but not required) (ARUP supply #47808). Available online through eSupply using ARUP Connect™ or contact ARUP Client Services at (800) 522-2787. (Min: 4 slides) Protect paraffin block and/or slides from excessive heat.

Storage/Transport Temperature: Room temperature or refrigerated. Ship in cooled container during summer months.

Remarks: Include surgical pathology report.

Unacceptable Conditions: Specimens fixed or processed in alternative fixatives (alcohol, Prefer) or heavy metal fixatives (B-4 or B-5).

Frozen specimens. **Decalcified specimens.**

Stability (collection to initiation of testing): Ambient: Indefinitely; Refrigerated: Indefinitely; Frozen: Unacceptable