



TEST CHANGE

Disaccharidase in

Tissue 3002037, DISACT

Specimen Requirements:

Patient Preparation:

Collect:

~~Two 5 mg biopsies of small bowel tissue. Use DISACT (Disaccharidase) Tissue collection kit (ARUP Supply #64011) available online through eSupply using ARUP Connect(TM) or contact Client Services at 800-522-2787. Biopsies of small bowel by endoscopy.~~

Specimen Preparation:

~~Use DISACT (Disaccharidase) Tissue collection kit (ARUP Supply #64011) available online through eSupply using ARUP Connect(TM) or contact Client Services at (800) 522-2787. Place one biopsy two 5 mg biopsies of tissue on the wall of each collection a small, tightly capped plastic tube without any supporting media or an ARUP Standard Transport Tube and freeze within 2 hours of collection. (Min: 2 (two) 5 mg biopsies).~~

Transport Temperature:

Frozen. Ship on dry ice.

Unacceptable Conditions:

~~Specimens placed on gauze, filter paper, or any other type of support media. Specimens Tissue preserved in formalin or in liquid.~~

Remarks:

Stability:

~~Ambient: Unacceptable; Refrigerated: Unacceptable; Frozen: 6 months~~

Methodology:

Quantitative Spectrophotometry

Performed:

Mon, Wed, Fri

Reported:

1-6 days

Note:

CPT Codes:

82657

New York DOH Approval Status:

This test is New York DOH approved.

Interpretive Data:

~~This test was developed and its performance characteristics determined by ARUP Laboratories. It has not been cleared or approved by the US Food and Drug Administration. This test was performed in a CLIA-certified laboratory and is intended for clinical purposes.~~

Deleted Cells

Reference Interval:

Component	Reference Interval
Lactase	Greater than or equal to 10.0 umol/min/g protein
Maltase	Greater than or equal to 100.0 umol/min/g protein
Palatinase	Greater than or equal to 9.0 umol/min/g protein
Sucrase	Greater than or equal to 25.0 umol/min/g protein