

New Test

3002581

Vaginosis Panel by TMA

VPAN TMA



Specimen Collection and Handling

Methodology: Qualitative Transcription-Mediated Amplification

Specimen Required: Patient Prep: Patient must be 14 years of age or older.

Collect: Vaginal specimen collected with pink swab from Aptima MultiTest Swab Collection kit (ARUP supply #55224 PK/50 or #55229 PK/10) available online through eSupply using ARUP Connect or contact Client Services at (800) 522-2787.

Specimen Preparation: Place swab in MultiTest Swab Specimen Transport Tube, break shaft at scoreline, then recap tube.

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Specimens in any transport media other than indicated above. Specimen in MultiTest swab transport media without a swab.

Stability (collection to initiation of testing): Ambient: 30 days; Refrigerated: 30 days; Frozen: 90 days

Reference Interval: Negative.

Interpretive Data:

See report

CPT Code(s): 87801, 87481, 87661

New York DOH Approved.

HOTLINE NOTE: Refer to the Test Mix Addendum for interface build information.