

TEST CHANGE

Cytokine Panel, TH2, Plasma

3002627, CYT TH2 P

Specimen Requirements:

Patient Preparation:

Collect: Green (lithium heparin).

Specimen Preparation: **CRITICAL FROZEN:** Separate plasma from cells ~~ASAP~~ within 2 hours of collection. Transfer 1 mL plasma to an ARUP **standard transport tube**. ~~Standard Transport Tube~~. (Min: 0.4 mL) **Freeze plasma within 30 minutes.**

Transport Temperature: CRITICAL FROZEN. Separate specimens must be submitted when multiple tests are ordered. Ship in ARUP **standard transport tube**. ~~Standard Transport Tube~~.

Unacceptable Conditions: Refrigerated specimens. Contaminated or heat-inactivated specimens.

Remarks:

Stability: After separation from cells: Ambient: 30 minutes; Refrigerated: Unacceptable; Frozen: 1 year

Methodology: Quantitative Multiplex Bead Assay

Note: Cytokine levels may demonstrate diurnal variation. For longitudinal comparison, it is recommended that cytokine levels be determined at the same time of day.

CPT Codes: 83520 x4

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Results are used to understand the pathophysiology of immune, infectious, or inflammatory disorders, or may be used for research purposes.

Reference Interval:

Test Number	Components	Reference Interval
	Interleukin 10, Plasma	5.3 pg/mL or less
	Interleukin 13, Plasma	5.3 pg/mL or less
	Interleukin 4, Plasma	2.5 pg/ml or less
	Interleukin 5, Plasma	2.1 pg/mL or less