

HOTLINE: Effective May 14, 2020

New Test **3002729** **Beta Amyloid Protein by Immunohistochemistry** **B AMYL IHC**
Available Now

Methodology: Immunohistochemistry
Performed: Mon-Fri
Reported: 1-3 days

Specimen Required: Collect: Tissue.

Specimen Preparation: Formalin fix (10 percent neutral buffered formalin) and paraffin embed specimen (cells must be prepared into a cellblock). Protect paraffin block and/or slides from excessive heat. Transport tissue block or 5 unstained (3- to 5-micron thick sections), positively charged slides in a tissue transport kit (recommended but not required), (ARUP supply #47808) available online through eSupply using ARUP Connect or contact ARUP Client Services at (800) 522-2787. (Min: 2 slides) If sending precut slides, do not oven bake.

Storage/Transport Temperature: Room temperature. Also acceptable: Refrigerated. Ship in cooled container during summer months.

Unacceptable Conditions: Specimens submitted with nonrepresentative tissue type. Depleted specimens.

Stability (collection to initiation of testing): Ambient: Indefinitely; Refrigerated: Indefinitely; Frozen: Unacceptable

Interpretive Data:

See Compliance Statement B: www.aruplab.com/CS

Note: This test is performed as a stain and return (technical) service only.

CPT Code(s): 88342

New York DOH Approved.

HOTLINE NOTE: Refer to the Test Mix Addendum for interface build information.