

HOTLINE: Effective April 20, 2020

New Test Available Now	3002776	COVID-19 IgG, Qualitative by CIA	COV19QUALG
----------------------------------	----------------	---	-------------------

Methodology: Qualitative Chemiluminescent Immunoassay
Performed: Sun-Sat
Reported: 1-5 days

Specimen Required: Collect: Serum separator tube (SST) or EDTA plasma.
Specimen Preparation: Separate serum or plasma from cells ASAP or within 2 hours of collection. Transfer 0.5mL serum or plasma to an ARUP Standard Transport Tube. (Min: 0.25 mL)
Storage/Transport Temperature: Refrigerated.
Remarks: Required: ARUP Standard Transport Tube for specimen submission (ARUP Item# 15824). Do not submit sample in original collection tube.
Unacceptable Conditions: Grossly hemolyzed, grossly icteric, or severely lipemic specimens.
Stability (collection to initiation of testing): Refrigerated: 1 week; Frozen: 1 month

Reference Interval: Negative

Interpretive Data: This test was developed and its performance characteristics determined by ARUP Laboratories. Testing was conducted in a CLIA certified laboratory. It has not been reviewed by the FDA. This test should not be used for screening of donated blood.

CPT Code(s): 86769

New York DOH Approved.

HOTLINE NOTE: Refer to the Test Mix Addendum for interface build information.