

**TEST CHANGE**

**Paraneoplastic Reflexive Panel**

3002929, PNS PAN2

**Specimen Requirements:**

**Patient Preparation:**

**Collect:** Serum ~~separator tube~~ **Separator Tube** (SST)

**Specimen Preparation:** Separate from cells ASAP or within 2 hours of collection. Transfer 3 mL serum aliquot to an ARUP standard transport tube. (Min: 1.0 mL)

**Transport Temperature:** Refrigerated

**Unacceptable Conditions:** Contaminated, heat-inactivated, hemolyzed, or lipemic specimens

**Remarks:**

**Stability:** After separation from cells: Ambient: 48 hours; Refrigerated: 2 weeks; Frozen: 1 month

**Methodology:** Semi-Quantitative Cell-Based Indirect Fluorescent Antibody ~~/~~ **Qualitative Immunoblot** ~~/~~ **Semi-Quantitative Indirect Fluorescent Antibody (IFA)**

**Performed:** Wed

**Reported:** 1-9 days

**Note:** Purkinje ~~c~~Cell (PCCA) antibody and ~~neuronal nuclear~~ **Neuronal Nuclear** (ANNA) antibody IgG are screened by IFA. If the IFA screen is indeterminate, then a Neuronal Nuclear Antibodies (Hu, Ri, Yo, Tr/DNER) IgG by Immunoblot will be added. If the IFA screen is positive at 1:10 or greater, then a PCCA/ANNA antibodies titer and Neuronal Nuclear Antibodies (Hu, Ri, Yo, Tr/DNER) IgG by Immunoblot will be added. Additional charges apply. If CV2 Antibody IgG Screen by IFA is positive, then CV2 Antibody IgG Titer by IFA will be added. Additional charges apply.

**CPT Codes:** 86255 x2; 84182 ~~x3~~x2; if reflexed, add ~~86256 and/or~~ 84182 x4; ~~if reflexed add~~ 86256 **per titer**

**New York DOH Approval Status:** This test is New York DOH approved.

**Interpretive Data:**

Refer to report

Reference Interval:

Test Number	Components	Reference Interval
	Neuronal Antibody (Amphiphysin)	Negative
	<u>Ma2/Ta Antibody, IgG by Immunoblot, Ser</u>	<u>Negative</u>
	Purkinje Cell/Neuronal Nuclear IgG Scrn	None Detected
	CV2 Ab IgG CBA-IFA Screen, Serum	Less than 1:100
	SOX1 Antibody, IgG by Immunoblot, Serum	Negative

**HOTLINE NOTE:** There is a component change associated with this test. One or more components have been added or removed. Refer to the Hotline Test Mix for interface build information.