

HOTLINE: Effective February 16, 2021

---

<b>New Test</b>	<b>3003311</b>	<b>Kratom (Mitragynine) - Screen with Reflex to Confirmation/Quantitation, Urine</b>	<b>KRATOM U</b>
-----------------	----------------	--	-----------------

**Methodology:** Qualitative Immunoassay/Quantitative Liquid Chromatography/Tandem Mass Spectrometry  
**Performed:** Varies  
**Reported:** 6-9 days

**Specimen Required:** Collect: Urine.

Specimen Preparation: Transfer 10 mL urine to ARUP Standard Transport Tubes. (Min: 1 mL)

**Test is not performed at ARUP; separate specimens must be submitted when multiple tests are ordered.**

Storage/Transport Temperature: Frozen. Also acceptable: Refrigerated.

Unacceptable Conditions: Urine from preservative tube

Stability (collection to initiation of testing): Ambient: 72 hours; Refrigerated: 2 weeks; Frozen: 6 months

**Reference Interval:** By report

**Note:** If screen is positive, then confirmation will be added. Additional charges apply.

**CPT Code(s):** 80307 if reflexed, add 80323

New York DOH approval pending. Call for status update.

**HOTLINE NOTE:** Refer to the Test Mix Addendum for interface build information.