

HOTLINE: Effective February 16, 2021

New Test **3003485** **Immunoglobulin G, CSF** **IGGCSF**

Methodology: Quantitative Immunoturbidimetry
Performed: Sun-Sat
Reported: Within 24 hours

Specimen Required: Collect: CSF.
Specimen Preparation: Centrifuge and separate to remove cellular material. Transfer 1 mL CSF to an ARUP Standard Transport Tube.
(Min: 0.5 mL)
Storage/Transport Temperature: Refrigerated.
Unacceptable Conditions: Grossly bloody or hemolyzed specimens.
Stability (collection to initiation of testing): Ambient: Unacceptable; Refrigerated: 1 month; Frozen: 6 months

Reference Interval: 0.0-6.0 mg/dL

CPT Code(s): 82784

New York DOH Approved.

HOTLINE NOTE: Refer to the Test Mix Addendum for interface build information.