

3003648

COVID-19 IgG (Spike), Semi-Quantitative by CIA

COV19G SQ

**Specimen Required:** Collect: Serum separator tube (SST). Also **acceptable:** lithium heparin.

Specimen Preparation: Separate serum or plasma from cells ASAP or within 2 hours of collection. Transfer 1 mL serum or plasma to an ARUP Standard Transport Tube. (Min: 0.5 mL)

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Specimens containing particulate material or otherwise obviously contaminated. Severely hemolyzed, heat-inactivated, **severely icteric** or lipemic specimens.

Stability (collection to initiation of testing): After separation from cells: Ambient: 72 hours; Refrigerated: 1 week; Frozen: 1 month (avoid repeated freeze/thaw cycles)