

TEST CHANGE

MLH1 Promoter Methylation 3004308, MLH1 PCR

3004308, MLH1 PCR	
Specimen Requirements:	
Patient Preparation:	
Collect:	Tumor tissue.
Specimen Preparation:	Tumor Tissue: Formalin fix (10 percent neutral buffered formalin) and paraffin embed tissue. Protect from excessive heat. Transport tissue block or 5 unstained 5-micron slides. Transport block and/or slide(s) in a tissue transport kit (ARUP Supply #47808) available online through eSupply using ARUP Connect(TM) or contact ARUP Client Services at 800-522-2787.
Transport Temperature:	Room temperature. Also Acceptable: Refrigerated. Ship in cooled container during summer months.
Unacceptable Conditions:	Less than 25 percent tumor. Specimens fixed in any fixative other than 10 percent neutral buffered formalin. Bone specimens submitted in non-EDTA decalcifier.
Remarks:	Include surgical pathology report. If multiple specimens (blocks or slides) are sent to ARUP, they must be accompanied by one of the following: an order comment indicating that the ARUP pathologist should choose the specimen most appropriate for testing (e.g., "Choose best block"), or individual orders for each sample submitted. A Pathologist Block Selection Fee (ARUP test code 3002076) will be added to orders that utilize the first option. If multiple specimens are sent to ARUP without a request for pathologist block/slide selection or individual orders, they will be held until clarification is provided.
Stability:	Ambient: Indefinitely; Refrigerated: Indefinitely; Frozen: Unacceptable
Methodology:	Real-Time Polymerase Chain Reaction //Fluorescence Resonance Energy Transfer (FRET)
Performed:	Varies
Reported:	7-12 days
Note:	
CPT Codes:	81288

Effective Date: October 20, 2025



New York DOH Approval Status:	This test is New York DOH approved.
Interpretive Data:	
Refer to report.	
Reference Interval:	

Effective Date: October 20, 2025

HOTLINE NOTE: There is a component change associated with this test. One or more components have been added or removed. Refer to the Hotline Test Mix for interface build information.