

HOTLINE: Effective September 21, 2021

New Test 3004316 NKX2.2 by Immunohistochemistry NKX2.2 IHC

Available Now

Methodology: Immunohistochemistry

**Performed:** Mon-Fri **Reported:** 1-3 days

Specimen Required: Collect: Tissue.

Specimen Preparation: Formalin fix (10 percent neutral buffered formalin) and paraffin embed specimen (cells must be prepared into a cellblock). Protect paraffin block and/or slides from excessive heat. Transport tissue block or 5 unstained (3- to 5-micron thick sections), positively charged slides in a tissue transport kit (highly recommended), (ARUP supply #47808) available online through eSupply using ARUP Connect or contact ARUP Client Services at (800) 522-2787. (Min: 2 slides) If sending precut slides, do not oven bake.

 $\underline{\underline{Storage/Transport\ Temperature:}}\ Room\ temperature.\ Also\ acceptable:\ Refrigerated.\ Ship\ in\ cooled\ container\ during\ summer\ months.$ 

<u>Unacceptable Conditions:</u> Specimens submitted with nonrepresentative tissue type. Depleted specimens.

Stability (collection to initiation of testing): Ambient: Indefinitely; Refrigerated: Indefinitely; Frozen: Unacceptable

## **Interpretive Data:**

This test was developed and its performance characteristics determined by ARUP Laboratories. It has not been cleared or approved by the US Food and Drug Administration. This test was performed in a CLIA-certified laboratory and is intended for clinical purposes.

**Note:** This test is performed as a stain and return (technical) service only.

**CPT Code(s):** 88342

New York DOH Approved.

**HOTLINE NOTE:** Refer to the Test Mix Addendum for interface build information.