

HOTLINE: Effective March 22, 2022

New Test Available Now SATB2 by Immunohistochemistry

SATB2 IHC



Immunohistochemistry Stain Form Recommended (ARUP form #32978)

Methodology: Immunohistochemistry

3004727

**Performed:** Mon-Fri **Reported:** 1-3 days

Specimen Required: Collect: Tissue or cells.

Specimen Preparation: Formalin fix (10 percent neutral buffered formalin) and paraffin embed specimen (cells must be prepared into a cellblock). Protect paraffin block and/or slides from excessive heat. Transport tissue block or 5 unstained (3- to 5-micron thick sections), positively charged slides in a Tissue Transport Kit (ARUP supply #47808) available online through eSupply using ARUP Connect<sup>TM</sup> or contact ARUP Client Services at (800) 522-2787. (Min: 2 slides). If sending precut slides, do not oven bake.

Storage/Transport Temperature: Room temperature. Also acceptable: Refrigerated. Ship in cooled container during summer months. Remarks: IMMUNOHISTOCHEMISTRY ORDERING AND SUBMISSION DETAILS: Submit electronic request. If you do not have electronic ordering capability, use an ARUP Immunohistochemistry Stain Form (#32978) with an ARUP client number. For

additional technical details, contact ARUP Client Services at (800) 522-2787.

<u>Unacceptable Conditions:</u> Specimens submitted with nonrepresentative tissue type. Depleted specimens.

Stability (collection to initiation of testing): Ambient: Indefinitely; Refrigerated: Indefinitely; Frozen: Unacceptable

## **Interpretive Data:**

This test was developed and its performance characteristics determined by ARUP Laboratories. It has not been cleared or approved by the U.S. Food and Drug Administration. This test was performed in a CLIA-certified laboratory and is intended for clinical purposes.

## Note:

This test is performed as a stain and return (technical) service only.

**CPT Code(s):** 88342

New York DOH Approved.

**HOTLINE NOTE:** Refer to the Test Mix Addendum for interface build information.