

HOTLINE: Effective May 16, 2022

New Test 3004788 Pancreatitis Panel (CFTR, CTRC, PRSS1, SPINK1), Sequencing PANC NGS



Additional Technical Information



Patient History for Pancreatitis Testing

Methodology: Massively Parallel Sequencing/Sequencing

Performed: Varies **Reported:** 3 weeks

Specimen Required: Collect: Lavender or pink (EDTA) or yellow (ACD solution A or B).

Specimen Preparation: Transport 3 mL whole blood. (Min: 2 mL)

Storage/Transport Temperature: Refrigerated.

Unacceptable Conditions: Serum or plasma; grossly hemolyzed or frozen specimens; saliva, buccal brush, or swab; FFPE tissue.

Stability (collection to initiation of testing): Ambient: 72 hours; Refrigerated: 1 week; Frozen: Unacceptable

Reference Interval: By report

Interpretive Data:

Refer to report.

This test was developed and its performance characteristics determined by ARUP Laboratories. It has not been cleared or approved by the U.S. Food and Drug Administration. This test was performed in a CLIA-certified laboratory and is intended for clinical purposes.

Counseling and informed consent are recommended for genetic testing. Consent forms are available online.

Note: Genes Tested: CFTR, CTRC, PRSS1, SPINK1

CPT Code(s): 81223; 81404; 81405

New York DOH approval pending. Call for status update.

HOTLINE NOTE: Refer to the Test Mix Addendum for interface build information.