

HOTLINE: Effective May 2, 2022

New Test **3005003** **Matrix Metalloproteinase-9 (MMP-9)** **MMP-9**

Methodology: Quantitative Enzyme-Linked Immunosorbent Assay
Performed: Varies
Reported: 3-10 days

Specimen Required: Collect: Serum Separator Tube (SST).
Specimen Preparation: Transfer 1 mL serum to an ARUP Standard Transport Tube and freeze immediately. (Min: 0.5 mL)
Test is not performed at ARUP; separate specimens must be submitted when multiple tests are ordered.
Storage/Transport Temperature: **CRITICAL FROZEN**
Stability (collection to initiation of testing): Ambient: Unacceptable; Refrigerated: Unacceptable; Frozen: 2 months

Reference Interval: By report

CPT Code(s): 83520

New York DOH Approved.

HOTLINE NOTE: Refer to the Test Mix Addendum for interface build information.