

NEW TEST – Available Now

[Click for Pricing](#)

ST2, Soluble

3006111, ST2 SOL

Specimen Requirements:

Patient Preparation:

Collect: Plain red. Also acceptable: Serum separator tube (SST).

Specimen Preparation: Transfer 1 mL serum to an ARUP standard transport tube. (Min: 0.2 mL) Test is not performed at ARUP; separate specimens must be submitted when multiple tests are ordered.

Transport Temperature: Frozen. Also acceptable: Refrigerated.

Unacceptable Conditions:

Remarks:

Stability: Ambient: 72 hours; Refrigerated: 1 week; Frozen: 3 months

Methodology: Quantitative Enzyme-Linked Immunosorbent Assay

Performed: Varies

Reported: 3-11 days

Note:

CPT Codes: 83006

New York DOH Approval Status: This test is New York DOH approved.

Interpretive Data:

Reference Interval:

By report

HOTLINE NOTE: Refer to the Hotline Test Mix for interface build information.