

NEW TEST - Available Now

Click for Pricing

Annexin A1 by Immunohistochemistry

3016561, ANNEX IHC			
Specimen Requirements:			
Patient Preparation:			
Collect:	Tissue		
Specimen Preparation:	Formalin fix (10 percent neutral buffered formalin) and paraffin embed specimen (cells must be prepared into a cellblock). Protect paraffin block and/or slides from excessive heat. Transport tissue block or 5 unstained (3- to 5-micron thick sections), positively charged slides in a Tissue Transport Kit (ARUP supply #47808 highly recommended) available online through eSupply using ARUP Connect or contact ARUP Client Services at 800-522-2787 (Min: 2 slides). If sending precut slides, do not oven bake.		
Transport Temperature:	Room temperature. Also acceptable: Refrigerated. Ship in cooled container during summer months.		
Unacceptable Conditions:	Tissue or cells not processed and placed in a paraffin block; serum, blood, or other body fluids; tissue not mounted on positively charged slides.		
Remarks:			
Stability:	Ambient: Indefinitely; Refrigerated: Indefinitely; Frozen: Unacceptable		
Methodology:	Immunohistochemistry (IHC)		
Performed:	Mon-Fri		
Reported:	1-3 days		
Note:	This test is performed as a stain and return (technical) service only.		
CPT Codes:	88342		
New York DOH Approval Status:	Specimens from New York clients will be sent out to a New York DOH approved laboratory, if possible.		
Interpretive Data:			



Reference Interval:

Test	Components	Reference Interval
Number		

HOTLINE NOTE: Refer to the Hotline Test Mix for interface build information.